### Finalidade

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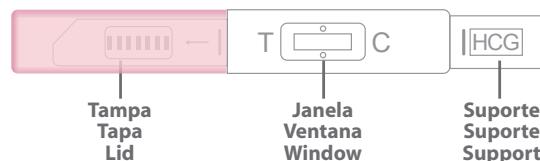
### Purpose

The Confirme<sup>®</sup> Teste Compact is a safe, easy and fast test for the detection of hCG in urine,any time of day, from the first day of missed period, but you can perform it up to 4 days before according to the follow study results (please check previous results in the item Question and Answers).

### Principle

The Confirme<sup>®</sup> Teste Compact is a qualitative immunoassay employing the sandwich method (5-6) to detect the human chorionic gonadotropin hormone (hCG) in urine.The membrane is impregnated with monoclonal anti-hCG antibodies in test area and with polyclonal antibody anti hCG in the control area. During the test, if urine has the presence of hCG, it reacts with the anti hCG monoclonal antibody colored conjugate pre-fixed to the membrane. The mixture goes through the membrane by capillary action to react with the anti - hCG fixed and forms a pink line at the test region. The presence of this pink line indicates a positive result, while its absence indicates a negative result.Regardless of the presence of hCG, the mixture or only the conjugate, continues to rise for the control area, reacting with the antibody in the control area, forming a pink line. The presence of this pink line in the control area serves to demonstrate that the volume and flow of the sample was adequate and appropriate, so as the functionality of the reagents.

### Content of the Confirme<sup>®</sup> Teste Compact:

- An aluminum envelope containing:  
A device (MIDSTREAM HCG) and a desiccant bag (do not use the test) Note: The desiccant is for storage of the test.
- An instruction for use.

### Warnings and Precautions

The package containing the device should be opened only at the time of use. The package contains the device and a bag of desiccant that should be discarded. The test should be kept at a temperature of 2 °C - 30 °C. It is a self-test for guidance without diagnostic purposes. For "in vitro" diagnostic use. Not intended for internal use. Follow carefully the procedure for testing.

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### Testing

Open the envelope and remove the test holding the support test (Figure 1).

Remove the lid. (Figure 2)

Point the absorbent membrane towards your urine stream. Maintain this position for 10 seconds or until the urine penetrates the membrane absorber.(Figure 3).

(Take care that the stream of urine does not point to the window. The urine can not be absorbed by the window. If this occurs the test can be invalidated).( Figure 4)

Put the cover back and rest the test on a flat surface. Wait 5 minutes to read the result.

Or if you prefer, collect urine in a clean and dry recipient, dip the absorbent tip until the urine is absorbed. Remove from the urine, wait 5 minutes to read the result. (Figure 5)

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## Quality Assurance

### How does Confirme® Teste Compact works?

Confirme® Teste Compact detects the hCG (HumanChorionic Gonadotropin) hormone, which is present in the urine of pregnant women in larger quantities.

Confirme® Teste Compact is accurate?  
Confirme® Teste Compact has an accuracy of 99.9%.

I have to use first morning urine?  
No, you can use Confirme® Teste Compact at any time of day, but the first morning urine may have a higher concentration of hCG and therefore be more likely to give a positive result.

When should I get tested?  
You can perform the Confirme® Teste Compact on the first day you miss your menstrual period, but you can perform it up to 4 days before according to the follow study results:

Number of days before missed period day	% of samples of pregnant women positive results
1º Day	98%
2º Day	97%
3º Day	88%
4º Day	56%

In case you perform the test in the mentioned conditions, it means the hormone concentrations not detectable yet and test should be repeated after 1º missed period.

### Some medication can cause false results?

All drugs containing hCG (Human Chorionic Gonadotropin) can give false result.

### If the result is positive?

If the result is positive, see your doctor.

### If the result is negative?

If the result is negative, it means that the hormone hCG was not detected. If your menstrual period does not come within a week, repeat the test and if negative again, and you do not menstruate, see your doctor.

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1.Batzer, FR Fertility, Vol 34, 1, 1980. - 2.Catt, Kjet al, JL, J. Endocrinol, Metab, Vol 40, 537, 1975. - 3.Braunstein et al., Vol 126, 678, 1976. - 4.Lenton, EA, et al.R. Fertility and Sterility, Vol 37, 773, 1982. - 5.Wide, L., Acta Endocrinol, 41 (Suppl.70) 75, 1962. - 6.Dawood, MY, et al.Ob Gyn, Vol 126, 678, 1976. - 7.Braunstein, GD, et al: Ann. Inter. Med. Vol 78, pp. 39-45, 1973. - 8.Engavil, E. Method in Enzymology, Vol 70, pp. 419-439, 1980. - 9.Uotila, M., et al, Immunol. Methods, Vol 42, 11, 1981. - 10. Data on file, Cal-Test Diagnostics, Inc. Chino, CA 91710, USA. - 11.Meyer, HM, Jr., "Statement to be include in Directions for Use of Anti-HCG Serum" Memorandum, DEW, BOB, november 2, 1973.